


# NEW Rocket® Rectal Solution Set

The NEW Rocket® Rectal Solution Set has been recently redesigned in conjunction with a leading UK Spinal Cord Injuries Centre to provide a simpler and safer method of injecting rectal solutions to aid bowel movement in patients with neurogenic bowel dysfunction such as that following spinal cord injury.

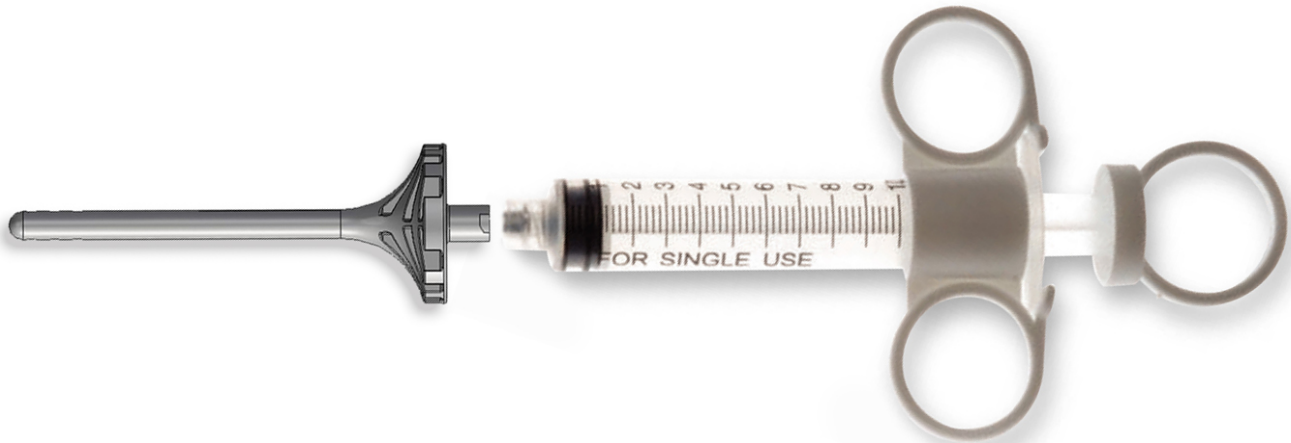

- **NEW MOULDED NOZZLE** produced in medical grade polyurethane for the correct combination of softness and rigidity for reliable, safe administration of a rectal solution. Suitable for self administration or by a personal carer.
  - **LARGE 3 RING SYRINGE** for easier handling and control
  - **INTEGRAL FLANGE** improves safety by preventing accidental over-insertion of the injector. Nozzle dead space from syringe connection to proximal hole: 0.7ml
  - **WIDELY RECOMMENDED:** The Rocket® Injector has been providing safe, reliable service to our many home users, Spinal Cord Injury Centres and Community Nurses throughout the UK since 2003. This new design is the result of user research aimed at improving safety and ease of use.
  - **REPLACES ORIGINAL STYLE:** This improved design will replace the original Rocket Rectal Injector from October 2010.
  - **EASY ON-LINE ORDERING:** The new design is now available from our discrete on-line ordering service as well as hospital and Spinal Cord Injury Centre use.
- Wash your hands. Open the pack and remove the syringe and nozzle assembly.
  - Attach the nozzle to the syringe by gently screwing the white moulding into the top of the syringe.
  - Locate and turn clockwise until lightly tightened.
  - Draw up the required dose of your rectal injection solution. The rectal injector can also be used to improve the administration of micro-enemas.
  - KY Jelly® may be used to lubricate the nozzle if required.
  - Using the most convenient position (e.g. standing, sitting on the toilet or lying on your side with your knees drawn up), gently insert the nozzle into the rectum 3-4cm. Slowly depress the plunger of the syringe to expel the contents into the bowel, wait a few seconds, then withdraw the nozzle completely.
  - Discard the combined syringe and nozzle assembly and wash your hands.
  - See pack leaflet for full Instructions for Use. Use only on the advice of your doctor, nurse specialist, community nurse or continence advisor who will assess your suitability for this device.

Ordering Information	Code
<b>Rocket® Rectal Solution Set</b> for the administration of rectal solutions Supplied individually wrapped in cartons of 10 units	<b>R56215-M</b>

