

Rocket® Amnioscope Set

complete with



Amnilume™ Disposable Light Source

The Rocket® Amnioscope Set complete with Amnilume™ Disposable Light Source is designed for those occasions when limited cervical dilatation requires a smaller, 23mm amnioscope.

The set comes complete with a sterile, self contained, battery powered high intensity light source designed to permit up to 5 FBS samplings.

- **SAFETY:** Saves struggling to locate a suitable reusable amnioscope with light source, fibre cables and reusable connectors when time is precious.
- **ALWAYS AVAILABLE:** Reliable lighting in all FBS conditions. One Amnilume is designed to provide sufficient illumination for up to 5 FBS samples in the same patient.
- **SIMPLICITY:**
 1. Remove the amnioscope and Amnilume™ from the pack
 2. The Amnilume™ Disposable Light Source locates into a slot at the rear of the amnioscope.
 3. Angle the Amnilume™ slightly downwards and slide forward until it locates securely into the slot (above).
 4. The action of pushing the device into position will automatically activate the light.
 5. Complete the sampling procedure and remove the endoscope.
 6. Remove the light source by turning over the amnioscope and pushing the locating tab gently with the thumb until detached. This action turns the device OFF and readies it for use for the next sampling.
 7. Repeat from Stage 3 for each sampling. Ensure the device is turned OFF between samplings.

Duty cycle: 5 x 15min ON periods with 5mins OFF between each period.

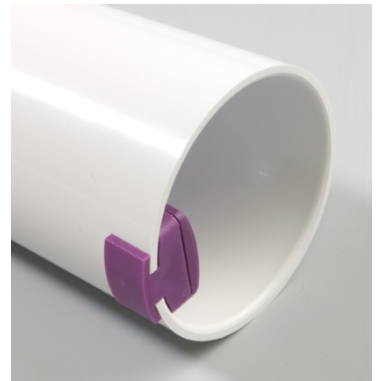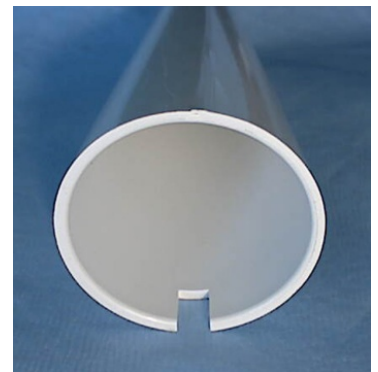

Description	Code
Rocket® Amnioscope Set with Amnilume™ Disposable Light Source 23mm OD. Supplied sterile in cartons of 10 sets for single patient use	R57017

